
C € 0197 ISO 13485 CERTIFICATION


AVAILABLE SIZES

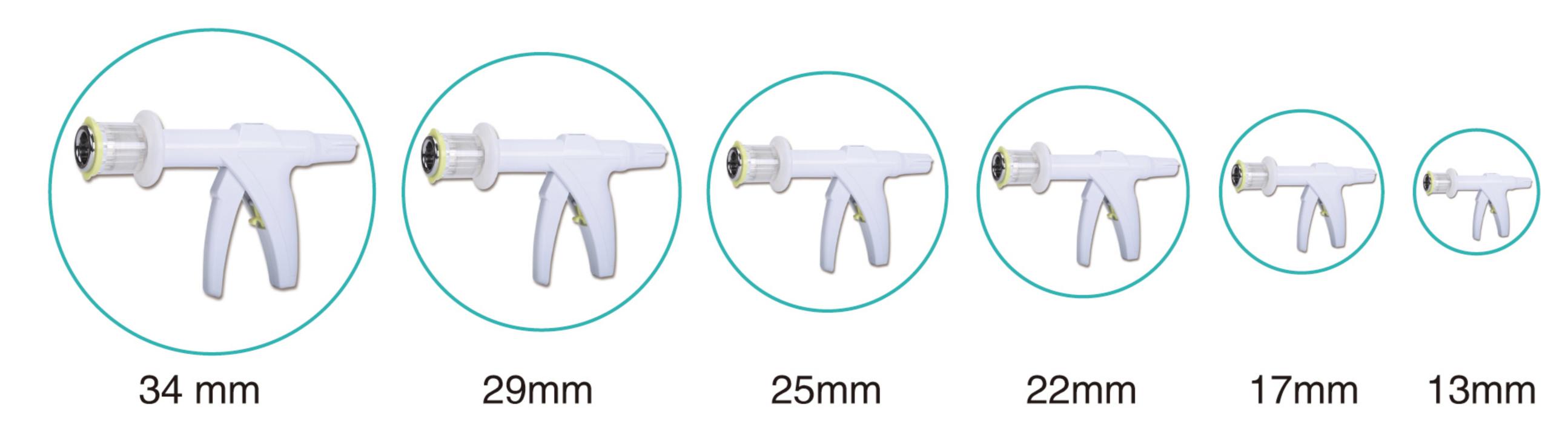


COMPARISION

	Traditional surgery	Circumcision surgery (Various inner+outer caliper)	CYS Circumcision Staplers For Single Use
Operation person	Surgeon and assistant	Surgeon and assistant	Sole operation
Surgery duration	Moren than 30 min	More than 15 min	3–5 min
Hemorrhage during the surgery	A lot	Few	Extremely few. Almost no
Hematoncus or not	Common	Occasionally	Rare
Stitches removal after surgery	Remove the stitches 10 days after the surgery	Remove the caliper one week after the surgery. The incision is prone to cracking after removal of the caliper.	No removal of stitches required. The stitches will drop of automatically.
Limitation duration	It is prohibited to take movement for two weeks and sexual for two months.	It is prohibited to take movement for two weeks and sexual for two months.	It is prohibited to take movement for two weeks and sexual for two months.
Incision observation	Not flat. When bleeding again, the patient need to go back to the hospital for secondary suturing. Long healing duration required.	Relatively flatter. When bleeding again, the patient need to go back to the hospital for secondary suturing. Long healing duration required.	Very flat. Rapid healing.
Postoperative nursing	Need to go back to the hospital for several times for dressing nursing.	Need to go back to the hospital for several times for dressing nursing.	Determine to go back to the hospital for dressing nursing or not ten days after the surgery.
Acceptability	The adult can easily accept the surgery. Children can hardly accept due to a great amount of anesthesia.	It is only applicable to the adult. It is hardly accepted by both of the adult and children.	Both the adult and children can easily accept the surgery.
Physicians opinion	It is very trouble, long duration, serious hematoma. Pain, not easy to accept.	Swelling, angina. Must go back to the hospital one week after the surgery to remove the caliper, not easy to accept.	Safe, painless, short surgery duration, efficient and convenient. It is pleasant to accept the surgery.


OPERATION PROCEDURES

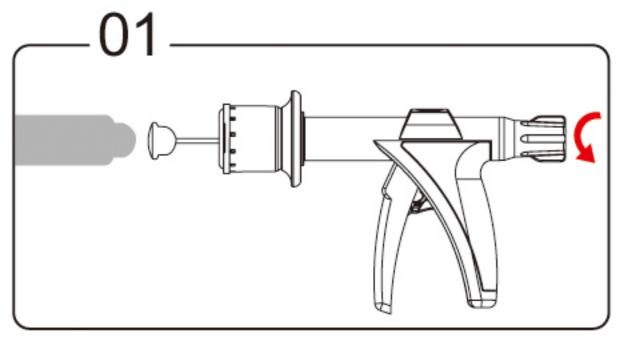


Turn left the rotating knob, pull out the penis cabin and staple cap.

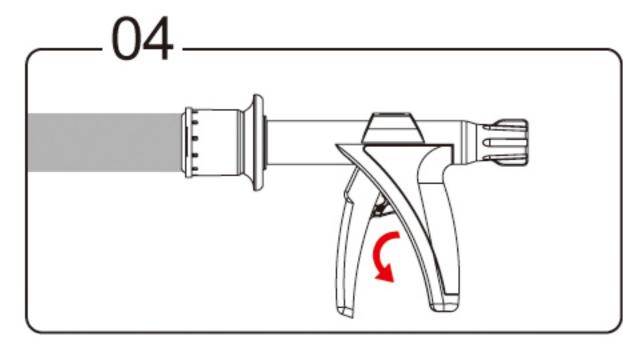


Pull down the safety clasp and ready for firing.

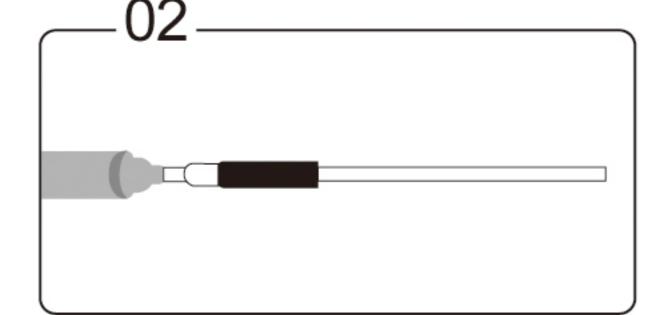


Push the penis cabin into the foreskin in a slant way. To tie the foreskin with a purse string suture (or band).

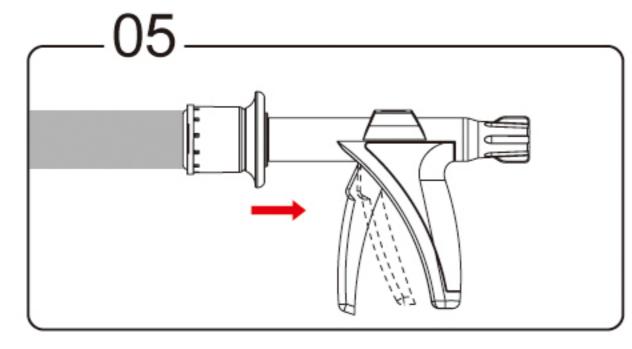

Press and hold the handle for a while, to make the foreskin to be stapling and suturing.

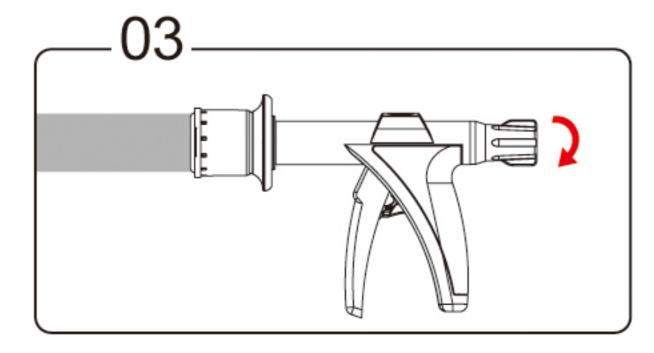

Insert the pull road into stapler, turn right the rotating knob smooth tightly to check the indicator in the window.

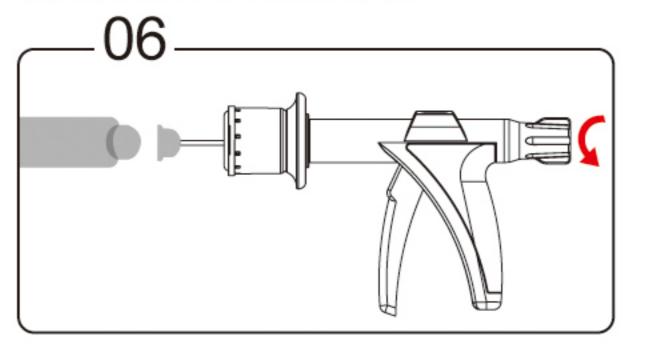

Turn left the rotating knob smoothly until the penis is separated from the penis cabin naturally.

CONTACT US



Changzhou Medical Bioengineering Co., Ltd

No. 117, West Chenjiatou, Heping Village, Zhenglu Town, Tianning District, Changzhou City, 213114 Jiangsu, China TEL: +86 519 88966750

Email: changsheng@china-changsheng.com
Web: www.china-changsheng.com





GYS circumcision

CIRCUMCISION STAPLERS FOR SINGLE USE

WORTY FOR YOUR FOCUS

- **M** HUMANIZED
- **M** HANDY
- **SAFETY**



Changzhou Medical Bioengineeg Co., Ltd



Type C

OVERVIEW

CIRCUMCISION has a history of 4000 years, it used to remove the excess or diseased foreskin from the penis, achieve a kind of operation that shows penis head adequately, as the treatment solution of phimosis or excess foreskin, also is the basic operation that a male urologist must master.

BENEFITS FOR CIRCUMCISION

FOR CHILDREN

- M Eliminate phimosis hazards
- M Avoid the influence for the penis' upgrowth
- M Reduce the probability of infection of penis, foreskin and urinary tract

FOR ADULT MEN

- M Improve the penis local health
- M Reduce the foreskin and glans inflammation
- M Reduce the risks of penile cancer incidence
- ☑ Reduce the risks of syphilis, acuteness wet wart, genital herpes, the risk of HIV/AIDS and other sexually transmitted diseases
- MReduce the risks of spouse or sexual partner, human papilloma virus caused by cervical cancer incidence of vaginal infections

HUMANIZED

- 1. The clip—on jointed pull rod, separated from the device to connect with the penis.
- 2. The penis cabin adopts gradually tighten connection with thread tap, easy for operation.
- 3. Unique holes in penis cabin is useful for separation of the penis and penis cabin after operation.

HANDY

- 1. The Silicone finger rest is designed in the device, useful for one person operation.
- 2. The finger rest will support the placed device to reduce the risks of device's pollution.
- 3. The jointed rotating knob connected with the body, do not drop, lost, easy for operation.

SAFETY

- 1. The safety clasp will keep locked to reduce the wrong operation before firing the staples.
- 2. Indicator in visible window is helpful for precise position cutting status of foreskin's thickness.
- 3. Unique design wide body narrow foot staples, benefit for staple's falling off after operation.

IDEAL

- 1. The pistol design circumcision device, easy and soft for firing, meet the ergonomics design.
- 2. Visible host shell is helpful for by checking the moving cutting blade and precise cut tissues to ensure its safety.
- 3. The silicone ring is designed on top of staples, as a blocker between staples and foreskin, aviod the staples is embedded in skin while healing.

Features: (Type B)

- The clip—on jointed pull rod, silicone finger rest is useful for one man operation.
- Indicator in visible window is helpful for precise position cutting status of thickness of foreskin.
- Visible host shell is helpful to checking the moving cutting blade and cut tissues to ensure its safety.
- Unique design staples are easy for stapling and falling off.
- Reduce the workload of doctors and improve operation efficiency.

Features: (Type A)

- Replacing traditional circumcision, it's safe and economical.
- Quick threaded design reduces the number of turns and shortens the operation time.
- Sharp ring cutting blade to ensure that the foreskin is cut off once.
- Unique holes in penis cabin is useful for separation of the penis and penis cabin after operation.
- Suturing and cutting at the same time, to cut foreskin by adjusting knob as doctor's experience.

